closomat®

DAS ORIGINAL DUSCH-WC.

CLOSOMAT® Lima Plus + Palma Plus

BEDIENUNGSANLEITUNG
MODE D'EMPLOI
OPERATING INSTRUCTIONS

LIEBE KUNDIN

LIEBER KUNDE

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DEAR CUSTOMER

Many thanks for choosing a CLOSOMAT® product. You will not regret your choice of the original among the douche-WCs, because the unique CLOSOMAT® douche concept will open up a new and higher quality of life for you, based on the stimulating interaction between natural bodily hygiene, physical well-being and health. It makes use of the force of the elements of water and air, and applies these in the correct sequence, at the right place and in the optimal quantities.

The anal region, which is frequently stressed by frequent sitting and a diet low in roughage, is thoroughly washed, gently massaged by the water spray and, following the well-known "Kneipp" theory, the circulation of the blood is stimulated by the initially warm and then slowly cooler douche jet. Here lies the secret of the lasting feeling of freshness that is experienced. Furthermore, as an additional important hygienic aspect, the hands remain free, and the drastic reduction in the use of toilet paper also makes a contribution to the overall positive balance.

You will certainly want to profit from all the advantages of your new system as quickly as possible. And its reliable function should also provide you with undisturbed pleasure throughout its long service life. You are therefore well advised to carefully read through these instructions. On the following pages, you will find many tips and suggestions regarding the correct use of your new CLOSOMAT®. The table of contents will help you to quickly find the required information. We particularly recommend the important instructions starting on Page 53, as well as the Maintenance, Care and Cleaning chapters.

Should you have any questions, please do not hesitate to call our Customer Service.

In order that we can be sure that everything is in order with the installation and function of your CLOSOMAT®, we would ask you to immediately fill in the reply section of the guarantee card completely, and to return it to us by post.

We now wish you a successful entry into your new, personal age of spa-like well-being, whereby, after a short period of accustomisation, you will certainly never want to do without this daily feeling of freshness again, a feeling that is shared by all other CLOSOMAT® users.

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Type label

The type label is located on the right side of the CLOSOMAT unit just under-neath the lower edge of the cistern cover. It contains the following information:

- 1** trademark (logo)
- 2** device type
- 3** nominal voltage
- 4** line frequency
- 5** power
- 6** power input
- 7** boiler volume
- 8** boiler pressure "standby"
- 9** manufacturer
- 10** date of manufacturing
- 11** symbol for marking of electric devices as to 2002/96/CE
- 12** symbol "read instructions"
- 13** symbol "pay attention to the accompanying documents"
- 14** CE label
- 15** protection class
- 16** serial number

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Contents	Page	Contents	Page
Putting your CLOSOMAT® into service	11	Alternative operating possibilities	31
Switching on main switch	11	Single-push operation	31
Functional check and demonstration of WC	11	Boiler economy control	33
Switching off the WC if not used for a long period	13	Operating accessories	33
Switching on the WC again	13	Maintenance, care and cleaning	35
Use of the CLOSOMAT®	15	Maintenance	35
Normal operation as douche-WC	15	Quality is good – control is better	35
Guests using CLOSOMAT®	15	Changing the activated carbon filter	37
1. Sitting	15	Care and cleaning	39
Temporary disconnection of odour extractor	15	Seat	39
2. Douche	17	Warm air outlet	41
Health benefits of the douche	19	Flush rim	41
Individually adjustable increase of douche pressure	21	Nozzle	43
Changing the douche jet basic pressure	21	Action in event of faults	45
Stimulation of bowel function	23	Douche does not function	45
3. Drying	25	Douche jet irregular	45
Premature interruption of drying	27	Warm air has unpleasant smell	47
Recommended alternatives for complete drying	27	Warranty	47
Special operation as normal toilet	29	Technical data	49
Flushing when used standing of for a power failure	29	Electrical data	49
		Water system	51
		Important information	53
		For reliable use	53
		Important points for care and maintenance	57
		Customer service addresses	60

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Putting your CLOSOMAT® into service

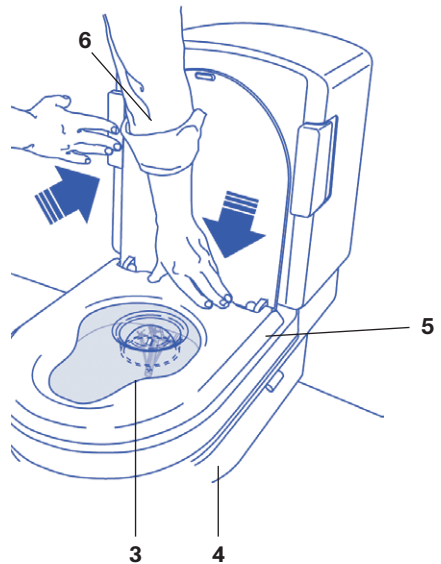
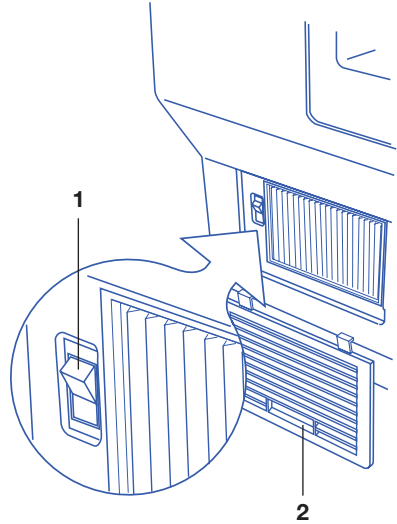
Switching on main switch

- Set main switch **[1]** (bottom left of WC behind filter cover **[2]**) to “On” (Illustration shows “off-position”).
- The main switch is normally left permanently “On” except when the CLOSOMAT® is not used for some time.

Functional check and demonstration of WC

The WC functions can be actuated without anyone sitting on the seat for the functional check and to demonstrate operation of the WC to other persons:

- Place the transparent checking cover **[3]** supplied between porcelain **[4]** and seat **[5]**: This screens the douche jet (illustration).
- Press one of the two push-keys **[6]** with one hand.
- With the other hand simultaneously press down the seat **[5]** at the rear (weight on seat hinges):
The douche nozzle moves from the initial position to the douche position and the douche jet begins.
- Load seat hinges further and release push-key; the douche nozzle returns to its initial position and the warm air drying is started automatically.

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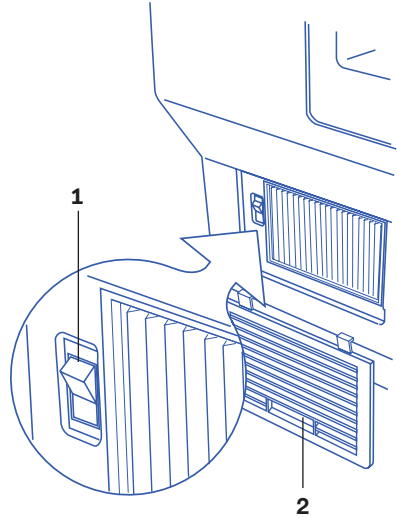
Switching off the WC if not used for a long period

If not used for longer than one week (e.g. holiday absence), energy is saved and scale deposits reduced by switching off the CLOSOMAT®.

It is essential to switch off if the WC is not used for more than two weeks.

- Set main switch **[1]** (bottom left of WC behind filter cover **[2]**) to "Off" (Illustration shows "off-position").


Switching on the WC again

- Main switch to "On" **[1]**

The following procedure is recommended to flush the system with fresh water if not used for a long period:

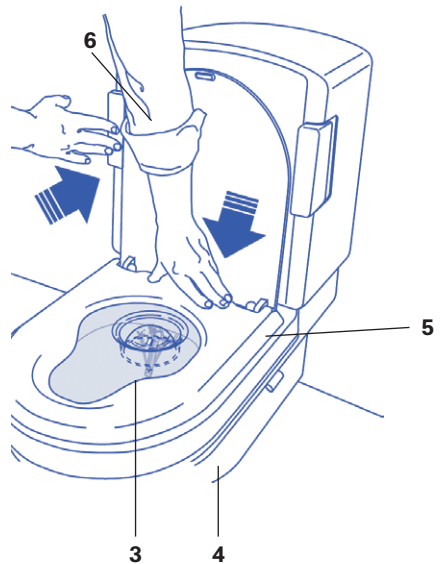- Press one of the two push-keys **[6]** without putting weight on the seat: Normal toilet flushing takes place.
- Press a push-key again and flush a second time.
- Place the transparent checking cover **[3]** supplied between porcelain **[4]** and seat **[5]** (screening of douche jet).
- Press one of the two push-keys **[6]** with one hand.
- With the other hand simultaneously press down the seat **[5]** at the rear (weight on seat hinges). The douche nozzle moves from the initial position to the douche position and the douche jet begins.
- Operate douche for approx. 30 seconds.
- Release push-key: the douche nozzle returns to its initial position.

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Use of the CLOSOMAT®

Normal operation as douche-WC

- Main switch should always be "On".
- Guests using CLOSOMAT® for the first time. To avoid unpleasant surprises, guests should be familiarized with the principal functions of the CLOSOMAT® before using it.

Frequency of use

The WC requires a heating time of three minutes after use before warm douche water is again available.

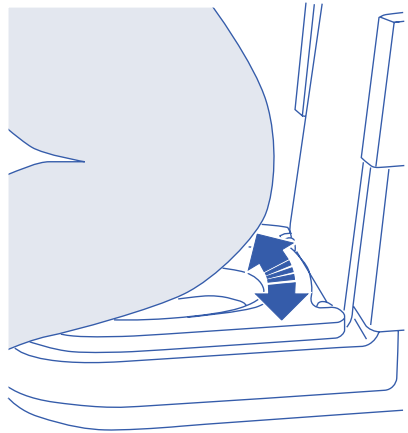
1. Sitting

- Sit on the seat upright and relaxed: the weight on the seat actuates the automatic odour extraction process. Bad air is removed from the pan and cleaned by an odour-absorbing activated carbon filter.
- Ensure that the rear zone of the seat is covered by the buttocks.

Temporary disconnection of odour extractor

If no odour extraction is required:

- Wait until the sound of air is heard from the odour extractor.
- Shift weight of body briefly forwards: odour extractor is switched off (weight removed from seat hinges).
- Sit normally again.

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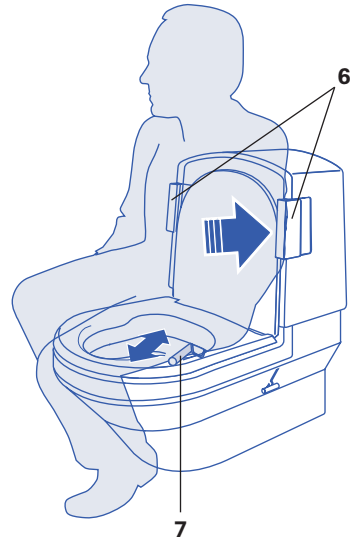
2. Douche

- Remain seated after evacuation of the bowels.
- Press one of the two elbow push-keys **[6]** continuously with the left or right elbow: the WC is flushed and the douche nozzle **[7]** moves from the initial position to the douche position. The warm douche jet starts after a brief cool jet.
- Keep elbow push-key pressed and enjoy the gentle thorough cleansing – first with water at body temperature and then gradually cooler.
A prolonged douche increases the benefits to health and therefore the douche duration is not preprogrammed, but can be individually selected by continuous operation of the elbow push-key.
- Sit as relaxed and mobile as possible: this makes the cleansing and massage more pleasant and effective.
- Release the elbow push-key only after about 20 seconds when the water flow becomes cool. The douche nozzle immediately returns to its protected initial position and will thereby be sprayed and cleaned with a targeted jet of water.
The warm air drying (according to section 3. Drying) starts automatically.

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Health benefits of the douche

The initially warm water washes cleanly and opens the skin pores. These are then closed by the subsequent cooler water. The massage effect is refreshing and stimulates the blood circulation, which is in any case sluggish in the anal region. A more prolonged douche with cold water provides in a natural way the traditionally proven Kneipp effect recommended by many physicians.

This form of anal cleansing provides a welcome preventive measure against numerous anal disorders. Thorough cleanliness without friction is indispensable for therapeutic reasons, particularly for problems with haemorrhoids or eczema. This is often the only way to achieve lasting and longed for relief if ointments and other medicaments fail. Sufferers know from experience: dry wiping with paper is not only extremely painful, but also causes continuous re-infection.

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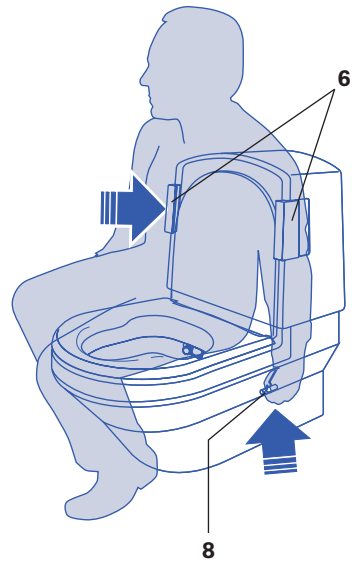
Individually adjustable increase of douche pressure

- Continue to press one of the two elbow push-keys **[6]** during the normal douche process described above.
- At the same time move the fully adjustable lever **[8]** on the left-hand side of the WC upwards slightly; the douche jet becomes stronger.
- Move the lever up until the desired douche jet pressure is set.

Returning to normal douche:

- Release lever slowly; the pressure of the douche jet returns automatically to the permanently set initial pressure.

Changing the douche jet basic pressure:

The basic pressure set at the factory can be increased or decreased by our service technician or your installer according to your requirements.

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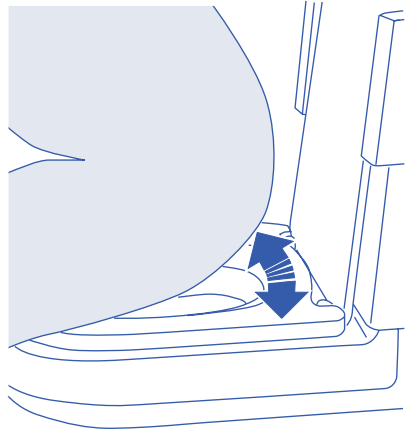
Stimulation of bowel function

(douche without immediately following drying)

The function of the bowels can be stimulated in the event of sluggishness by using the CLOSOMAT® douche. This can often initiate evacuation of the bowels in an uncomplicated and pleasant way. The same applies to a possible complete evacuation if only a partial evacuation has initially taken place.

In both cases the undesired supply of warm air after the douche is prevented as follows:

- Move the body weight forwards briefly (to take weight off seat): blower switches off.
- Resume normal sitting: odour extractor is switched on.

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3. Drying

- Remain seated after the douche: the warm air drying starts automatically. The WC switches back to odour extraction after about 3 minutes.

Drying – but not drying out

The electronically controlled variable reduction of dryer temperature prevents the skin drying out: the initially higher temperature is found pleasant on wet skin. After one minute, as the skin gradually dries, the temperature is reduced in several steps. The anal area is still not fully dry after the full duration of warm air drying. This may seem somewhat strange at first, but after a little use, you will no longer wish to be without the advantages of this ingenious, convenient system. Its gentle function provides daily well-being, hygienic safety and a lasting feeling of freshness.

Complete drying if medically prescribed (for example, if antibiotics are being taken!)

Where residual moisture is not required, complete drying as described in the section “Recommended alternatives for complete drying” on page 27 can be performed.

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Premature interruption of drying

If the warm air drying is subjectively found too long, it can be interrupted at any time as required:

- Stand up: removal of weight from the seat ends the drying process.
- Complete drying according to the following alternatives:

Recommended alternatives for complete drying

- The most environmentally-friendly procedure is the use of a terry towel hung near the CLOSOMAT®. It is in fact only necessary to dab off a few clean drops of water.
- Another alternative is to dab with toilet paper, which should preferably be disposed of in the waste basket rather than by separate flushing.

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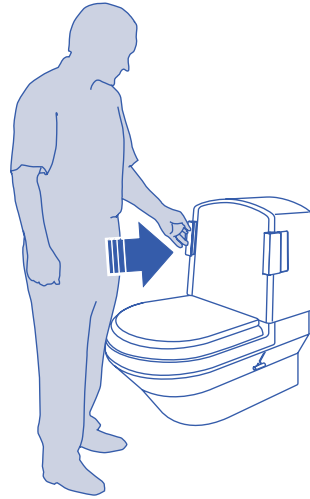
Special operation as normal toilet

Flushing when used standing of for a power failure

If there is no weight on the seat, pressing a push-key only actuates normal flushing. The CLOSOMAT® can therefore also be used at all times as a normal toilet by unaccustomed guests. This function is not even affected by a possible power failure.

For hygienic reasons, however, we also recommend all male users to sit down while urinating, because a CLOSOMAT® as well as a normal toilet is not suitable as a urinal.

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Alternative operating possibilities

Single-push operation

Normally the CLOSOMAT® will continue to operate the douche as long as one of the two elbow keys remains pressed. If, however, the functional sequence should take place automatically after a short impulse on one of the keys, with an adjustable douche period (up to ca. 45 sec.), the service technician can activate this mode. After having started the process, the running douche can be stopped intentionally by a second impulse on a key.

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Boiler economy control

Normally the CLOSOMAT® keeps the douche water ready at body temperature in a well insulated boiler. If the economy program is activated (by service technician), the boiler begins to heat up only when somebody sits down. Then the user must wait for ca. 2–3 minutes until douche water is available at body temperature. This mode is as well suitable for units that are only used occasionally.

Operating accessories:

Various operating accessories are offered to increase convenience or as an aid for handicapped persons. The range extends from pneumatic foot or hand switches, to infrared remote control.

Our customer service will gladly advise you (Tel. numbers on page 60).

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Maintenance, care and cleaning

- **Maintenance**

The CLOSOMAT® requires very little maintenance. The best precondition for long, fault-free operation is regular and frequent use, particularly where the water contains lime. It is unnecessary to install a descaling system just for the CLOSOMAT®, but with very hard water a central system of this kind protects the entire sanitary equipment and appliances in the home. Consult your plumber in this regard.

The douche nozzle is designed so that the holes cannot become calcified.

- **Quality is good – control is better!**

If you wish to assist actively, however, in reducing the risk of subsequent major repairs, we recommend you to complete and return the reply section of the warranty card. Our customer service will then contact you later to suggest servicing variants.

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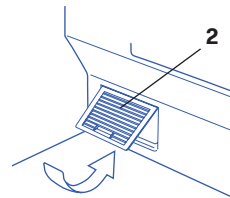
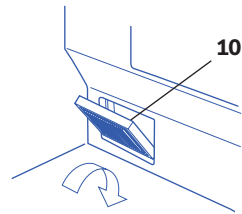
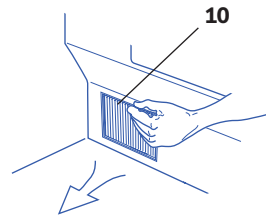
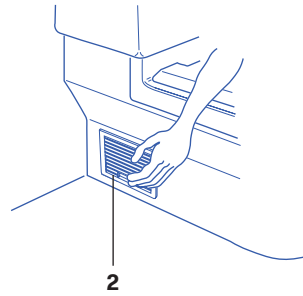
• Changing the activated carbon filter

It is easy to change the activated carbon filter in the odour extraction unit. The filter wears out and should therefore be replaced once or maximum twice a year depending on frequency of use.

- Remove filter cover **[2]** at bottom left of unit:
The main switch and filter cartridge **[10]** can be seen.
- Withdraw used filter cartridge.
- Insert new filter cartridge.
- Replace filter cover.

Order: Palima activated carbon filter
Art. No. 02-0920-00

The used filter can be discarded without reservation with the household waste.

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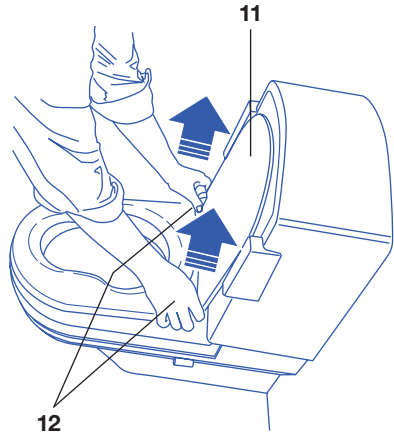
• Care and cleaning

The CLOSMAT® should basically be cleaned by hand like a normal WC. The pan of high-grade sanitary porcelain can be cleaned with all commercially available WC cleaners and scale removers (Never use any cleaning agents with chlorine, such as Javel water or bleach). The openings below the “closed” flush rim of the bowl must be kept free from calcium deposits in order to maintain the full rinsing performance. When purchasing cleaning agents ensure in general that they are environmentally-compatible and use them as sparingly as possible. Do not use WC deodorants, either suspended from the flushing rim or in the cistern. Make inquiries from our customer service concerning useful tips and utensils.

Seat

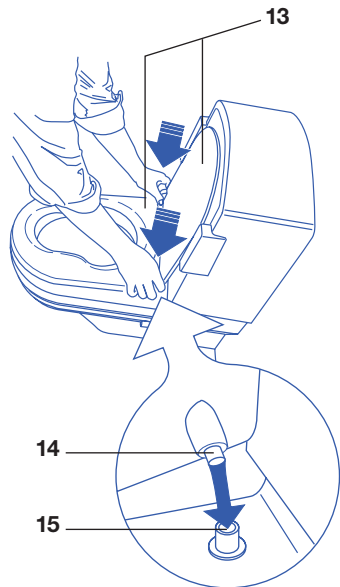
The seat is of insertable design to simplify cleaning around the seat hinge. It can be removed in one operation.

Removing the seat:

- Open cover [11].
- Grip inner side of hinge hear [12] with both thumbs.
- Hold seat with fingers and draw vertically upwards evenly with a slight jerk.

Refitting the seat:

- Place seat [13] horizontally on pan.
- Ensure that bolts [14] enter the sleeve holes [15].
- Ensure by pressing briefly on both rear cover corners that the seat has engaged correctly.

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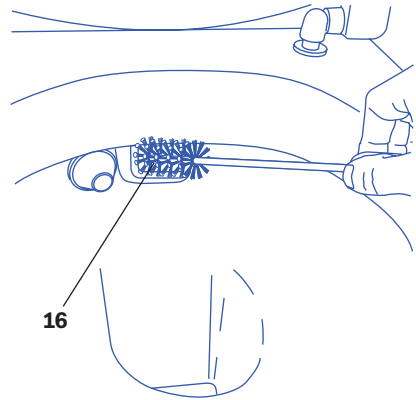
The seat and casing elements are coated with durable, acid-resistant, two-component varnish. These surfaces should be cleaned with a soft cloth or sponge with water and well-diluted, commercially available general purpose cleaning agents.


Warm air outlet

Preliminary note: a CLOSOMAT® as well as a normal toilet is not suitable as a urinal. We therefore recommend all male users to sit down while urinating for hygiene reasons. This prevents the warm air opening of the CLOSOMAT® becoming excessively soiled.

If the warm air duct **[16]** should nevertheless need cleaning occasionally, this can easily be performed – even when wet – with a suitable round brush (it is best not to wait until the warm air has an unpleasant odour).

Flush rim

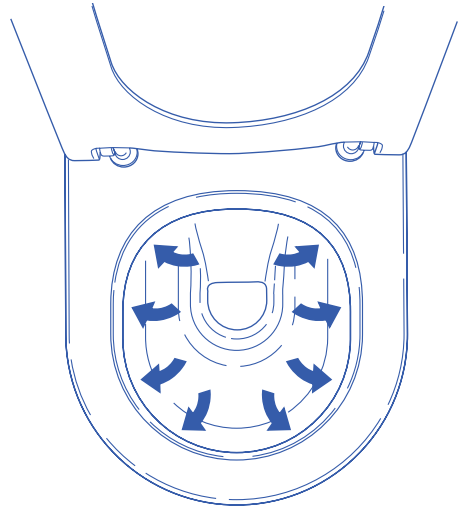Ensure that the holes under the „closed“ flush rim are kept free from lime deposits, so that the full flushing effect is maintained.

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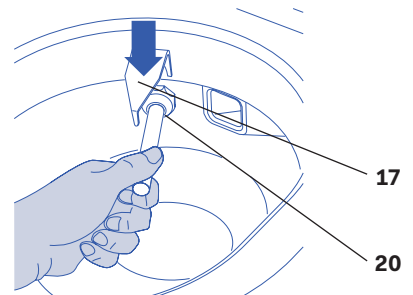
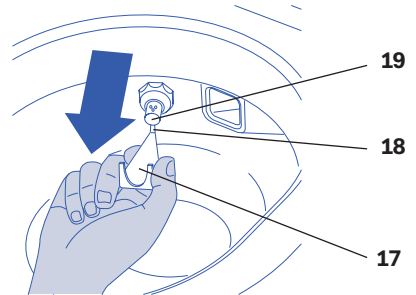
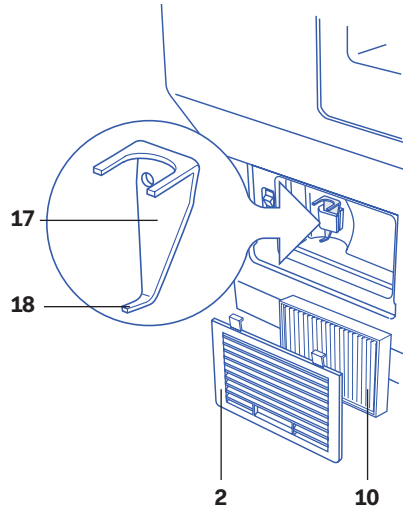
Nozzle

The design of the extending and retracting douche nozzle prevents it being soiled in both the rest and active positions. After every douche operation, the nozzle will be sprayed and cleaned by a targeted jet of water while retracting. The following procedure should be adopted if access is nevertheless required to the nozzle:

- Remove filter cover **[2]** at bottom left of the unit:
A small special tool **[17]** can be seen in a holding clamp behind the filter cartridge **[10]**.
- Remove tool from the holding clamp and insert with pointed side **[18]** in nozzle from below as shown in the diagram.
- Withdraw nozzle **[19]** with special tool **[17]** against the spring pressure as far as the stop and hold with the free hand.
- Turn tool round and insert the forked side in the annular groove **[20]** of the nozzle from above as shown in the diagram. This locks the nozzle.
- Clean nozzle.
- Hold nozzle in one hand, remove the special tool and slowly return the nozzle to its initial position.
- Place the special tool back in its holder.

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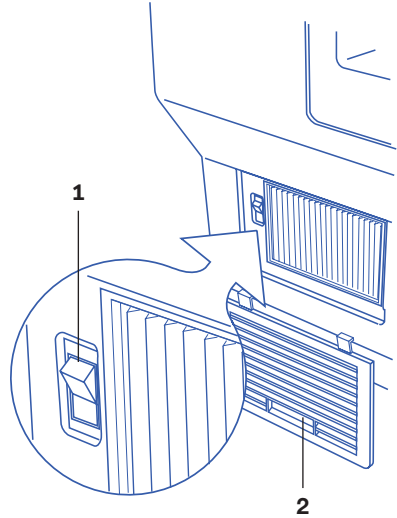
Action in event of faults

Douche does not work

- Check whether main switch **[1]** (bottom left of unit behind the filter cover **[2]**) is at "On" (Illustration shows "off-position").
- Place the transparent checking cover **[3]** supplied between porcelain **[4]** and seat **[5]**.
- Press seat **[5]** down at the rear with one hand (weight on seat hinge).
- Press one of the two push-keys **[6]** with the other hand.


If douche still does not work:

- Check fuse in power distribution box.
- Possibly press Reset button on earth leakage circuit breaker.
- Press one of the two push-keys with one hand. With the other hand simultaneously press down the back of the seat (weight on seat hinge).

If the douche does not work despite presence of supply voltage:

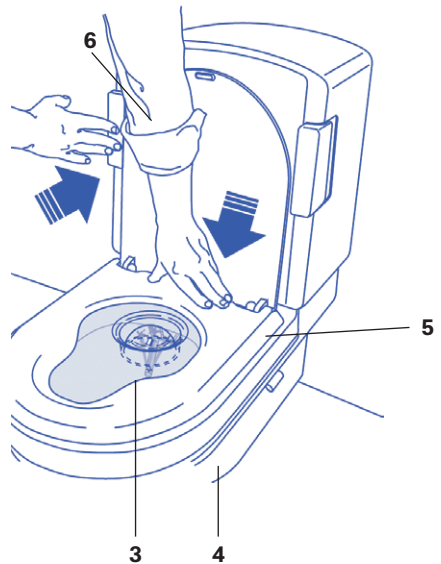- Check whether the cistern fills again rapidly after flushing (max. 1 min) (Water supply sufficient?).

Douche jet irregular

- Check whether water supply is available or if it takes long for the cistern to fill (max. 1 minute).

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Warm air has unpleasant smell

- In completely new units or following prolonged standstill it is normal for odours to be produced temporarily by dust burning on the heating wires.
- Possible soiling of the warm air duct should be eliminated (as described on page 41/ Warm air outlet).

If the procedure described does not produce the desired effect or if other irregularities are found, request assistance:

- from the sanitary fitter for water or drainage faults (blocked outlet, poor flushing, water only trickling into cistern, water leaking into pan or onto floor).
- from the CLOSOMAT® customer service for all other faults.

In this case the following details are useful for rapid clarification:

- Type and manufacturing number of CLOSOMAT® (according to warranty card)
- Approximate year of commissioning
- When was the last maintenance performed (easily determined if the last maintenance bill is kept).
- What exactly is not functioning?

Warranty

The warranty conditions effective can be found on the warranty card provided with the unit. If the card cannot be found, the relevant customer service office will be glad to help you (Customer service telephone numbers on page 60).

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Technical data

SH: Seat height *

440 mm seat height for CLOSOMAT® Lima Plus corresponds to an outlet height of (220 mm Minimum outlet height = 210 mm).

Electrical data:

Rated voltage: 230 V
Conductor capacity: 10 A
Rated consumption: 1300 W (water heating 800 W, air heating 350 W)

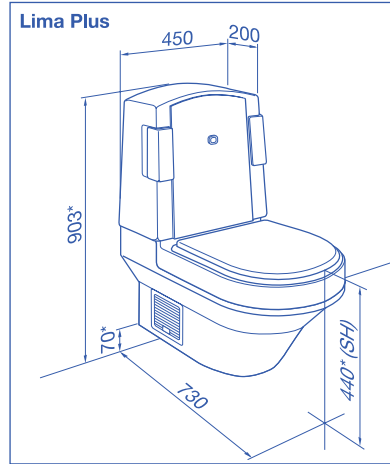
Energy required:

- Stand-by consumption: 0.3 kWh approx. per 24 hrs.
- Per cycle usage: 0.07 kWh approx. (this amount of energy is consumed by a 1000 W hair dryer in four minutes).

Control and monitoring:

- Fully-electronic, with additional mechanical safety temperature limiters
- Built unit main switch at bottom left behind filter cover.

The units are constructed and tested in accordance with current European safety guidelines for low-voltage appliances.

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Cold water connection only

Performance:

- Volume flow: min. 7 litres per minute
- Pressure: no special requirements owing to built-in pump

Cistern capacity:

Min. 6 to max 9 litres.

The Palma and Lima models are designed as 6-litre systems and set for this at the factory. (Your fitter can set the filling volume for any other possible conditions.)

Boiler:

- Volume: 1.7 litres approx.
- Stand-by temperature: 36–38°C approx. (can be set by service technician)
- Pressure:
 - Reset position: unpressurized
 - Working pressure: approx. 0.2 MPA (2 bar)
- Heating time after use: 3 mins. approx.

The boiler keeps a constant stand-by temperature, so that douche water at body temperature is available at all times. Alternatively: Boiler economy control (to be activated by service technician)

Douche jet force:

- Variable externally, by pulling adjusting lever
- Basic pressure can be set by service technician

The units are constructed in accordance with the guidelines currently effective in EC member states for drinking water supplies and waste water systems.

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Important information

The following points should be observed without fail for reliable use, satisfactory operation and a long service life of your CLOSOMAT®:

For reliable use

- Assume the correct sitting position when using the douche:
upper body upright, relaxed, back of seat opening covered by the buttocks.
- The head should never be lowered into the WC pan: there is danger of injury if the douche function is operated simultaneously.
- Never watch douche function without checking cover, in order to prevent inadvertent facial douche and water splashing everywhere.
- Use the CLOSOMAT® exclusively either as a normal WC or as douche WC to clean the anal area.
- Speak with your doctor about a precautionary examination of the bowels. Should it be necessary to examine the stool to identify any traces of blood or to take a stool sample, you are recommended to proceed as follows. When you go to the toilet, sit down the other way round to usual, with your face to the wall, such that the stool remains temporarily on the forward, flatter section of the bowl.
- The CLOSOMAT® douche should not be used following surgery or with open wounds in the anal or genital areas until healing is sufficiently far advanced. Seek the advice of your doctor.

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Important information

- Young children or handicapped and dependent persons must not sit unsupervised on the CLOSOMAT®.
- The WC is not a waste bucket. Do not discard solid articles in the pan (danger of porcelain breakage or blockage of outlet). Avoid large accumulations of WC paper. Sanitary towels, cotton-wool wads and the like must not be disposed of in the WC pan.

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Important points for care and maintenance

- The casing parts may only be removed by our service engineers or other authorized skilled persons. They conceal live electrical parts!
- The CLOSOMAT® must not be cleaned with running water outside the pan area.
- Do not use scouring agents or abrasive cloths or sponges for cleaning the CLOSOMAT®. Wash down varnished surfaces only with well-diluted domestic cleaning agents.
- Do not use steam cleaning units for cleaning the CLOSOMAT®.
- The seat unit and ceramic pan are sensitive to blows – do not allow seat and cover to drop when opening and closing.
- Never allow solvents, such as nail varnish or varnish remover, to come into contact with the CLOSOMAT®.
- Do not leave lighted cigarettes on the WC.
- The seat must be kept open while aggressive scale remover is taking effect in the pan.
- Never use any cleaning agents with chlorine, such as Javel water or bleach!
- The openings below the “closed” flush rim of the bowl must be kept free from lime deposits.
- Periodically check with finger pressure that the douche nozzle has run completely back into its rest position. If this is repeatedly not the case, please call our Customer Service.

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Important points for care and maintenance

- Do not use any “perfume stones” or other active agent dosage device on the rim of the pan or in the cistern.
- Protect the unit from direct sunlight. This can cause discoloration.
- The CLOSOMAT® must be protected from frost.

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Subject to alteration due to continuous adaptation and further development. Adaptations and developments or different designs may be the reason for the description not exactly matching the respective device design.

CLOSOMAT® has been the registered trademark for the world's first douche WC since 1957.

Service CLOSOMAT®

Schweiz/Suisse/Switzerland:

Closemo AG
Hardhofstrasse 21
CH-8424 Embrach
fon 0844 77 66 55
fax +41 (0) 44 866 86 81
office@closomat.ch
closomat.ch

Deutschland/Allemagne/Germany

CLOSOMAT GmbH Vertriebsbüro
Goethestrasse 12
DE-58675 Hemer
fon +49 (0) 23 72 935 98 88
fax +49 (0) 23 72 935 98 89
office@closomat.de
closomat.de